




Dario® Blood Glucose Monitoring Systems

User Guide

Applicable to meters models:

1021-04 Dario with Lightning connector (in US - Dario® LC)

1021-06 Dario with USB-C connector (in US -Dario® USB-C)

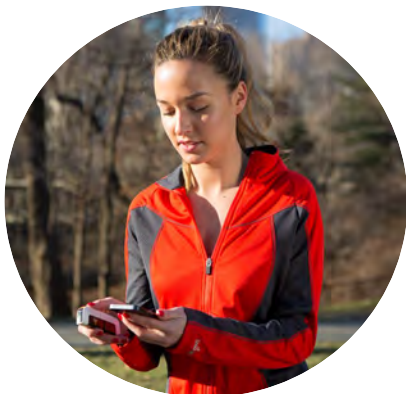

© 2025 LabStyle Innovation Ltd., ALL RIGHTS RESERVED

BASIC INFORMATION

Please read this user guide thoroughly. If you have questions, you can contact our customer support

- USA: toll-free Monday to Friday, 9:00AM – 9:00PM EST at 1-800-895-5921 or www.mydario.com
- Canada: toll-free at 1-800-461-0991 or www.mydario.ca/

Please contact your health care provider for questions or concerns outside of the normal customer support hours.



Manufacturer:

LabStyle Innovation Ltd.

5 Tarshish St. North Industrial Park

Caesarea 3079821, Israel

www.mydario.com

Dario® is a trademark of LabStyle Innovation Ltd.

United States: Product is covered by one or more patents and/or patent applications – the list is detailed at the Dario website at www.mydario.com.

Table of Content:

CHAPTER 1 INTRODUCTION AND IMPORTANT SAFETY

INSTRUCTIONS	6
GLUCOSE MONITORING FOR A MOBILE AGE	6
USER GUIDE SCOPE	7
INTENDED USE	7
SYSTEM LIMITATIONS	8
IMPORTANT SAFETY INSTRUCTIONS.....	9
BEFORE TESTING WITH THE DARIO SYSTEM	11
SAFETY DEFINITIONS	11
PRACTICES USED IN THIS USER GUIDE.....	12
GENERAL PRECAUTIONS.....	15
ADDITIONAL INFORMATION	17

CHAPTER 2 DARIO SYSTEM OVERVIEW AND FEATURES18

YOUR DARIO SYSTEM AT A GLANCE	18
DARIO COMPONENTS.....	19
COMPATIBLE PLATFORMS WITH THE DARIO BLOOD GLUCOSE MONITORING SYSTEM.....	21
DARIO SYSTEM KIT CONTENTS.....	22
GETTING STARTED WITH THE DARIO SYSTEM	24
ASSEMBLING THE DARIO SYSTEM FOR THE FIRST TIME	25
INSERTING THE TEST STRIP CARTRIDGE	25
INSERTING A LANCET	26
SETTING LANCET DEPTH (OPTIONAL)	28
THE DARIO APP	28
DOWNLOADING AND INSTALLING THE DARIO APP	29
CREATING YOUR ACCOUNT AND PERSONALIZING YOUR SETTINGS	31
SETTING YOUR PERSONAL AND DIABETES PROFILES	32

CHAPTER 3 CHECKING YOUR DARIO GLUCOSE METER34

DARIO CONTROL SOLUTION	34
CONTROL SOLUTION PRECAUTIONS	34
CONTROL SOLUTION TEST PRINCIPLE	35
CHEMICAL COMPOSITION	36
CHECKING YOUR GLUCOSE METER WITH DARIO CONTROL SOLUTIONS.....	36
REMOVING THE GLUCOSE METER FROM THE HOUSING	37
REMOVING A TEST STRIP FROM THE CARTRIDGE	38

PERFORMING A CONTROL SOLUTION TEST	39
CHAPTER 4 TESTING YOUR BLOOD GLUCOSE LEVEL	45
BLOOD GLUCOSE TESTING PRECAUTIONS.....	45
TEST STRIP PRECAUTIONS.....	46
FACTORS THAT INTERFERE WITH BLOOD GLUCOSE TESTING.....	48
INTERPRETING THE RESULTS.....	49
MEASUREMENT RANGE	50
FACTORS THAT MAY LEAD TO INACCURATE RESULTS	51
STEP ONE: INSERTING A NEW LANCET INTO THE LANCING DEVICE	52
STEP TWO: OPENING THE DARIO APP	52
STEP THREE (RELEVANT TO THE US MARKET ONLY): INSERTING THE SMART MOBILE DEVICE INTO THE DISPOSABLE COVER	53
STEP FOUR: REMOVING THE GLUCOSE METER FROM THE HOUSING	54
STEP FIVE: TESTING YOUR BLOOD GLUCOSE	55
STEP SIX: RETURNING THE METER TO THE DARIO HOUSING	62
TEST STRIP LIMITATIONS	62
CHAPTER 5 CLEANING AND MAINTENANCE	64
CLEANING AND DISINFECTION	64
STEP 1: CLEANING.....	65
STEP 2: DISINFECTION	65
CHAPTER 6 TEST STRIPS	70
PRODUCT DESCRIPTION AND THE PRINCIPLE OF THE USE.....	70
STORAGE AND HANDLING	70
REAGENT COMPOSITION (PER STRIP).....	71
PRECAUTIONS	71
SYSTEM LIMITATIONS	72
PERFORMING A TEST.....	73
UNDERSTANDING YOUR TESTING RESULT.....	73
UNEXPECTED RESULTS.....	73
CONTROL OF THE UNEXPECTED RESULTS	74
CAUSES OF UNEXPECTED RESULTS.....	74
CHECKING BY A CONTROL SOLUTION	75
CONTROL SOLUTION PROCEDURE.....	76
UNDERSTANDING YOUR CONTROL SOLUTION TEST.....	76
CHAPTER 7 TROUBLESHOOTING	78
TROUBLESHOOTING	79

PROBLEM SOLVING.....	81
TEST STRIP INSERTED BUT “PLEASE INSERT A NEW STRIP” KEEPS APPEARING ON THE SCREEN.....	81
YOU CANNOT PIERCE SKIN WITH LANCET.....	82
THE LANCET DOES NOT FIT CORRECTLY INTO THE LANCING DEVICE.....	82
CANNOT REMOVE THE USED NEEDLE FROM THE LANCING DEVICE.....	82
LANCING DEVICE DOES NOT OPERATE CORRECTLY.	83
LANCET DEPTH INDICATOR LEVER IS STUCK.	83
LANCING DEVICE RELEASE BUTTON DOES NOT WORK PROPERLY....	83
“STRIP IS USED AND SHOULD BE REPLACED” MESSAGE APPEARS IN THE DARIO APP.	83
I FORGOT MY DARIO PASSWORD.	83
MY SMART MOBILE DEVICE BATTERY DIED.....	84
CHAPTER 8 TECHNICAL INFORMATION AND SPECIFICATIONS	85
HOW THE GLUCOSE METER WORKS	85
COMPATIBILITY	85
PERFORMANCE	85
TECHNICAL SPECIFICATIONS	88
COMPLIANCE WITH ELECTRICAL AND SAFETY STANDARDS.....	89
DISPOSING OF USED PRODUCTS.....	93
INFORMATION SECURITY AND HIPAA	94
CHAPTER 9 WARRANTY, REPAIRS AND SUPPLIES	96
REPAIR POLICY	96
MANUFACTURER’S ONE YEAR WARRANTY	96
SUPPLIES AND ACCESSORIES.....	97
CONTACT CUSTOMER SUPPORT	98

CHAPTER 1

INTRODUCTION AND IMPORTANT SAFETY INSTRUCTIONS

Glucose Monitoring for a Mobile Age

Congratulations on your choice of the Dario System. The Dario System was created to make life easier for people with diabetes by providing the right tools, information, and support in your pocket. It is a smart phone-based monitoring system, featuring a pocket-sized meter with an integrated lancing device and test strips, combined with an easy-to-use mobile application.

Living with diabetes is a daily challenge. But with the Dario System in your pocket and the best support, you can thrive with diabetes.

This user guide describes how to properly setup and use your new Dario System. It is recommended that you read this guide thoroughly in order to familiarize yourself with the system and its many features.

User Guide Scope

The user guide is covering the use of the Dario LC and Dario USB-C Blood Glucose Monitoring System with the following meter models:

USA	Dario LC (model 1021-04) for use with selected iPhone mobile devices with Lightning connector
	Dario USB-C (model 1021-06) for use with selected Android and iPhone mobile devices with USB-C connector
Canada	Dario Meter (model 1021-04) for use with selected iPhone mobile devices with Lightning connector
	Dario Meter (model 1021-06) for use with selected Android mobile devices with USB-C connector

For the latest list of compatible devices please refer to customer support website at –

<https://www.dariohealth.com/supported-devices/>

Intended Use

The Dario with Lightning connector (in US - Dario® LC) and Dario with USB-C connector (in US -Dario® USB-C) Blood Glucose Monitoring System consists of the Dario Blood Glucose Meter, Dario Glucose Test Strips, Dario Glucose Control Solutions and the Dario App as the display component of the Dario with Lightning connector (in US - Dario® LC) and Dario with USB-C connector (in US -Dario® USB-C) Blood Glucose Monitoring System. The Dario with Lightning connector (in US - Dario® LC) and Dario with USB-C connector (in US -Dario® USB-C) Blood Glucose Monitoring System is intended for the quantitative measurement of glucose (sugar) in fresh capillary whole blood samples drawn from the fingertips. The Dario with Lightning connector (in US

- Dario® LC) and Dario with USB-C connector (in US - Dario® USB-C) Blood Glucose Monitoring System is intended to be used by a single person and should not be shared.

The Dario with Lightning connector (in US - Dario® LC) and Dario with USB-C connector (in US -Dario® USB-C) Blood Glucose Monitoring System is intended for self-testing outside the body (in vitro diagnostic use) by people with diabetes at home to monitor the effectiveness of diabetes control. The Dario with Lightning connector (in US - Dario® LC) and Dario with USB-C connector (in US -Dario® USB-C) Blood Glucose Monitoring System should not be used for the diagnosis or screening of diabetes or for neonatal use.

The Dario Blood Glucose Test Strips are for use with the Dario Blood Glucose Meter to quantitatively measure glucose (sugar) in fresh capillary whole blood samples drawn from the fingertip.

The Dario Glucose Control Solutions are for use with the Dario Blood Glucose Meter and the Dario Blood Glucose Test Strips to check that the meter and test strips are working together properly and that the test is performing correctly.

System Limitations

The Dario System should not be used for the following purposes:

- The diagnosis of, or screening for, diabetes.
- Testing the glucose levels of neonates (children younger than 4 weeks of age).
- Testing glucose levels of arterial or venous blood.

- Testing glucose from sites other than the fingertip.
- Testing patients who are critically ill, in shock, or dehydrated.

Certain health conditions, such as serious illness and dehydration, can cause false test results in glucose meters. You should not use this system to test your blood glucose if you are critically ill. If you are very sick or think you are dehydrated, or are urinating frequently, call your health care provider immediately.

If you are taking acetaminophen containing drugs (e.g., Tylenol) or Vitamin C (ascorbic acid) you may get inaccurate results with this system. If you have a disease or condition in which uric acid levels in your blood may be elevated (> 11 mg/dL), such as gout, you may get inaccurate results with this system. If you have very high levels of triglyceride (> 810 mg/dL), you may get inaccurate results with this system. This system should not be used when undergoing xylose absorption tests.

Do not use in oxygen rich environment and with flammable anesthetics and flammable agents.

Important Safety Instructions

The meter and lancing device are for single-patient use. Do not share them with anyone, including other family members. Do not use on multiple patients.

In Canada: the mobile App displays results in mmol/L. This is a set unit of measurement and cannot be changed.

All parts of the kit are considered bio-hazardous and can potentially transmit infectious diseases even after you have performed disinfection.

This device is not intended for use in healthcare or assisted-use settings such as hospitals, physician offices, or long-term care facilities because it has not been cleared by FDA for use in these settings, including for routine assisted testing or as part of glycemic control procedures. Use of this device on multiple patients may lead to transmission of Human Immunodeficiency Virus (HIV), Hepatitis C Virus (HCV), Hepatitis B Virus (HBV), or other bloodborne pathogens.

Keep the device and testing equipment away from young children and animals. Small items such as the test strips and lancets are choking hazards.

For important instructions for disinfecting your meter and lancing device, refer to **CHAPTER 5 CLEANING AND MAINTENANCE** on page 64.

The U.S. Food and Drug Administration (FDA) and the Centers for Disease Control and Prevention (CDC) have issued warnings and notifications regarding the risk of bloodborne pathogen transmission when blood glucose monitoring systems have been used by more than one individual. Refer to the following documents for more information:

- “FDA Public Health Notification: Use of Fingerstick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens :Initial Communication” (2010). You can find this document at www.fda.gov

- “CDC Clinical Reminder: Use of Fingertick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens” (2010). You can find this document at www.cdc.gov



Warning: To ensure accurate results, wash your hands with warm, soapy water and dry them before and after every test. Unwashed hands may lead to inaccurate results. Make sure there is no grease, oil, or lotion on the fingertip. Use lancets only once. Do not use any lancet or lancing device that has been used by another person.

Before Testing with the Dario System

Please read this user guide carefully and thoroughly before testing your blood glucose for the first time. These instructions provide the information that you need for safe use and handling of your Dario.

Incorrect use of this system may lead to an inaccurate glucose reading or damage to your device.

For any questions, contact customer support:

- USA: toll-free Mon to Fri, 9:00AM – 9:00PM EST at 1-800-895-5921 or www.mydario.com
- Canada: toll-free at 1-800-461-0991 or www.mydario.ca

Safety Definitions

Safety instructions are provided for the protection of Dario System users. This user guide includes three types of important messages: Warnings, Cautions, and Notes. Text marked with one of these symbols requires your attention to ensure safe and accurate use of your Dario System.

Warnings



A Warning indicates precautions you need to take to prevent possible risk of injury.

Do not proceed beyond a Warning message until you fully understand the conditions and have taken the appropriate preventive action.

Cautions



A Caution indicates precautions you need to take to prevent damage to your monitoring system.

Do not proceed beyond a Caution message until you fully understand and observe the indicated conditions.

Notes



Notes provide additional information for the purpose of clarification.

Practices Used in This User Guide

The following sections describe the formats and symbols used throughout this user guide, on the Dario System packaging and inserts, and on actual system parts.

Procedure Instructions


- Procedure instructions are clearly identified and presented as numbered steps.
- If relevant, a system response is written below the step in italics.

You should perform all procedures in the order of the steps provided.

























The examples and screen images used in this User Guide are provided as examples only (unless stated otherwise).



Words in:	Description
Bold	Names of menus, buttons, and screens.
<i>Italics</i>	System response that occurs as a result of pressing a button or performing a task.
<i>Bold Italics</i>	Emphasize important information.

Symbol	Description	Manufacturer
	Sterile Lancets Under Directive MDD 93/42/EEC	Beijing Ruicheng Medical Supplies Co., Ltd.

Symbols and Descriptions

The following table lists and describes the symbols that appear throughout this guide, on packaging or package inserts, or on the actual system parts.

Symbol	Description	Symbol	Description
	Dario logo		LabStyle Innovation Ltd. logo
	Consult instructions for use		Do not use if package is damaged
	In Vitro Diagnostic		Batch code
	Expiry date		Shelf life after you first opened the package
	Do not reuse		Temperature limitation
	Catalog number		Contains sufficient for <n> tests
	Manufacturer: LabStyle Innovation Ltd. 5 Tarshish St. Northern Industrial Park Caesarea 3079821, Israel		Patient information website
	Intended for self-testing		Sterilized using irradiation
	Keep away from sunlight		Keep dry
	Relative humidity limits		Warning – indicates precautions you need to take to prevent possible risk of injury
	Caution – indicates precautions you need to take to prevent damage to your monitoring system		Note – provide additional information for the purpose of clarification

Symbol	Description	Symbol	Description
	Importer		Distributor

General Precautions

This section provides information about potential hazards that may harm your Dario System or cause it to not function properly.

Inspect the Dario System prior to use. Avoid using any part of the system with broken seals or expired dates. Do not attach or use a component if it is damaged in any way, as it may not work properly.

- The Dario System is not water-resistant and should not get wet. If it does get wet, wipe it with a clean, dry cloth.
- Blood glucose testing with the Dario System should only be done in temperatures between 50°F-95°F (10°C-35°C) and relative humidity between 15-85%. Test results may not be correct if the room temperature and/or humidity are outside this range.
- Modifications to the Dario System not expressly approved by LabStyle Innovation Ltd. will void your warranty and could result in injury
- Avoid testing with your Dario System in altitudes higher than 10,000 feet (3,048 meters), as accuracy cannot be assured.
- Do not leave the meter in very hot or cold places. Do not leave it near a heat source (radiator) or in a car in hot or cold weather.

- Do not store the meter or test strips in an area where the humidity is outside of the range 10-90%, such as a bathroom or kitchen.
- Do not use in oxygen rich environment and with flammable anesthetics and flammable agents.



Use of Dario System adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.



Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.



Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Dario System hardware, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment

could result in inaccurate results or product malfunction.

Additional Information

For your safety, observe the following recommendations when monitoring your blood glucose using the Dario System:

- It is recommended that you have a backup testing method available. Failure to test could cause a delay in treatment decisions and lead to a serious medical condition. Examples of backup testing methods include a backup glucose meter device or testing by a laboratory. Ask your health care provider about other possible backup methods.
- Do not change your treatment based on a single result that is inconsistent with how you feel, or if you believe that your test result could be incorrect. The blood glucose reading displayed on your smart mobile device by Dario System should not be the only information you use to make decisions about your health. Always consult your health care provider to interpret and understand the measurement results.
- Never make changes to your diabetes control program or ignore physical symptoms without consulting with your health care provider.

CHAPTER 2

DARIO SYSTEM OVERVIEW AND FEATURES

This chapter provides an overview of the Dario System components and describes how to set up the Dario System for the first time.

Your Dario System at a Glance

The Dario System is a cloud-based diabetes management solution that combines software applications with a stylish all-in-one device. The all-in-one device features a lancing device, test strip cartridge, and a compact meter. Together with a smart mobile device, the Dario System puts everything in your pocket that you need to monitor and understand your diabetes. Our goal is to provide the right combination of tools, knowledge, and motivation that you need to succeed.

You can check your blood glucose with a small sample (0.3µl) of your blood and receive results within (6) six seconds.


You can check and track your blood glucose readings the same way you check everything else in life - on your mobile device.

The Dario Glucose Meter connects to smart mobile devices with an easy-to-use mobile App. The Dario mobile App provides real-time and actionable information so that you always know where you stand and what to do next.

The Dario System not only logs and tracks your glucose levels, but also food, activity, moods, and more. The Dario System can even alert your caregivers when you need special attention.

Dario Components

The Dario System components are shown in the image below and described in the following table.

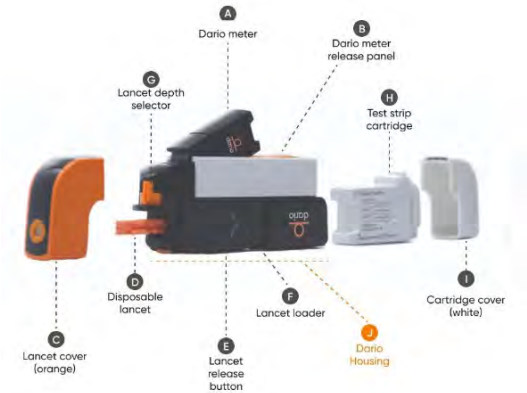

Component	Description
Glucose Meter (A) Dario meter (B) Dario meter release panel	<p>The miniature glucose meter consists of the following components:</p> <ul style="list-style-type: none"> • A connector that is plugged into the port of your smart mobile device • A strip port for insertion of the test strip <p>The glucose meter receives its power from the mobile device. It does not require batteries or any other electrical source. When not in use, the meter snaps into its slot in the housing for protected storage.</p>
(J) Dario Housing	<p>The Dario Housing (also referred to as the device housing or simply “housing”) is the main part of the Dario all-in-one device. It holds all the parts in place and consists of the lancing device, the meter slot, and the housing for the test strips cartridge.</p>
Lancing Device and Cover (C) Lancet cover (orange) (D) Disposable lancet (E) Lancet release button (F) Lancet loader (G) Lancet depth lever	<p>The lancing device is built into the housing and uses disposable Dario Lancets. The orange lancing device cover prevents the user from unintentional punctures. You must change lancets after each use. For more information about using the lancing device, refer to CHAPTER 4 TESTING YOUR BLOOD GLUCOSE LEVEL on page 45.</p>
(H) Test strip cartridge (I) Cartridge Cover (white)	<p>Each test strip cartridge holds 25 strips. Test strips are sold separately.</p> <p>The cartridge snaps into the modular Dario Housing.</p>

Component	Description
Test Strip	The Dario Test Strip is the only strip which is used with the Dario System. A new test strip is inserted into the test strip port of the glucose meter prior to each glucose measurement.
Lancets	The Dario System uses a 30G lancet. To replace the lancet, remove the orange lancing device cover, pull out the used lancet, and replace it with a sterile one.
Disposable covers (Applicable to the USA market only)	To avoid the risk of transmitting bloodborne pathogens, it is advised to use the disposable sleeve to cover your smart mobile device while taking the blood glucose test. When the test is concluded, remove the cover from your smart mobile device and dispose it together with the used lancet and test strip in accordance with the instructions provided by your health care provider for biohazardous waste.



It is recommended that you carry spare test strip cartridges and lancets in case of an emergency.

Compatible Platforms with the Dario Blood Glucose Monitoring System

The Dario LC and Dario USB-C Glucose Meters use smart phone technology to view and store unlimited numbers of glucose test results. You can also track other diabetes information and view graphs and trends. You can even share your diabetes management information with your health care provider.

The Dario LC System is compatible for use on iOS smart mobile devices. The Dario LC Glucose Meter currently works with the following Apple® iPhone® devices running iOS OS 13,14,15, 16 ,17 and 18.

iPhone devices: SE, 6s, 6s Plus, 7, 7 Plus, 8, 8 Plus, X, XR, XS, XS Max, 11, 11 Pro and 11 Pro Max, SE 2020, 12, 12 Mini, 12 Pro, 12 Pro Max, 13, 13 Pro, 13 Pro Max, 14, 14 Pro, 14 Pro Max, 15, 15 Pro, 15 Pro Max, 16, 16 Pro, 16 Pro Max.

The Dario USB-C System is compatible for use on Android and iOS smart mobile devices. The Dario USB-C Glucose Meter currently works with the following Android devices – Samsung Galaxy S8, S8+, S9, S9+, S10, S10+, S10e, S10 5G, S20, S20+, S20 Ultra, S21, S21+, S21 Ultra, S22+, S22 Ultra, S23, S23+, S23 Ultra, S24, S24+, S24 Ultra A50, A70, A31, A32, A41, A51, A51 5G, A51 5G UW, A52, A52 5G, A71, A71 5G, A71 5G UW, A72, S20 FE 5G. Note 8, 9, 10, 10+, 20, and 20 Ultra. LG, G6, G7, G8 and G8s. Google Pixel 3, 3 XL, 3a XL, 4, 4 XL, 5, 6, 6 Pro, 7, 7 Pro, 8, 8 Pro, 9, 9 pro. Running Android OS 9, 10, 11,12, 13 and 14.

The Dario USB-C Glucose Meter currently works with the following Apple® iPhone® devices: iPhone 15, 15 Pro, 15 Pro Max, 16, 16 Pro, 16 Pro Max.

Running iOS OS – 17, 18.

For the latest list of compatible devices please refer to customer support website at:

www.mydario.com/support/getting-started/

Dario System Kit Contents

As soon as you receive your Dario System Kit:

1. Carefully unpack the kit and verify that it contains the following parts:

- Dario System, which includes the lancing device and built-in blood glucose meter
- Dario 10 Sterile Lancets
- Dario 10 Disposable Covers (4.5"X 8.25") (available in the USA market only)
- Dario with Lightning connector (in US - Dario® LC) and Dario with USB-C connector (in US - Dario® USB-C) Blood Glucose Monitoring System Quick Guide
- Dario with Lightning connector (in US - Dario® LC) and Dario with USB-C connector (in US - Dario® USB-C) Blood Glucose Monitoring System User Guide
- Dario Test Strips (not provided within Dario welcome kit)

The Dario System also includes the following components that are sold separately:

- Dario Test Strips
- Dario Sterile Lancets
- Dario Disposable Covers (4.5"X 8.25") (available in the USA market only)
- Dario Control Solution Level 1 (Medium) and 2 (High)
- Disinfection disposable wipes:
 - USA - Super Sani Cloth® germicidal disposable wipes by PDI Healthcare
 - Canada - Germidice® by Germiphene



Once you remove the exterior packaging, you will find a white package with a tamper-proof

seal. If the tamper-proof seal is damaged or missing, it is recommended that you return this Dario System to where you purchased it, as it has been previously opened.

2. If anything is missing from the Dario System Kit, **Contact Customer Support**, for contact information please see page 98.
3. Dario System components and accessories (e.g., test strips, lancets, control solution, disposable covers) can be ordered from:
 - Your local Dario representative.
 - Selected pharmacies.
 - From the Dario App.
 - By contacting customer support:
 - USA: toll-free Monday to Friday, 9:00AM – 9:00PM EST at 1-800-895-5921 or www.mydario.com
 - Canada: toll-free at 1-800-461-0991 or www.mydario.ca

For details on product availability, refer to **Supplier and Accessories** on page 97.

Getting Started with the Dario System

Your Dario Glucose Meter works seamlessly with your smart mobile device. Simply download the Dario App on your compatible smart mobile device and it will automatically synchronize with the Dario device each time it is connected.

Your data is constantly synchronized to one place so that it is always with you on the go.

Your data is saved under your username on your smart mobile devices and on the Dario cloud. You

have easy access to your data through the Dario App and at www.mydario.com/.

Assembling the Dario System for the First Time

To set up the Dario System for the first time, you must complete the following procedures:

- Insert a test strip cartridge, as described in **Inserting the Test Strip Cartridge** on page 25
- Insert a lancet, as described in **Inserting a Lancet** on page 26
- Set the lancet depth, if desired, as described in **Setting Lancet Depth (Optional)** on page 28
- Download and install the Dario App to your smart mobile device, as described in **Downloading and Installing the Dario App** on page 29
- Set up the Dario App on your smart mobile device using the Dario App instructions or the instructions provided on the Dario website at:
USA: www.mydario.com
Canada: www.mydario.ca



Both ends of the Dario Housing have covers that are removable. The lancet cover is orange with a small opening. Be careful not to put the cartridge cover on the lancing end. This can damage the lancet and the lancing device.

Inserting the Test Strip Cartridge

To insert the test strip cartridge for the first time:

1. Open the white cover and remove the paper holder from the device.

2. Remove the new test strip cartridge from its pouch.
3. Hold the test strip cartridge with the side that opens facing up, as shown in the image below.



4. Insert the cartridge into the housing until it snaps into place.
5. Return the white housing cover by sliding it into place until it clicks, as shown in the image below.



! If the cover does not close properly, the cartridge may not have been inserted correctly. Ensure you have properly inserted the cartridge, as described in steps 3 and 4.

Inserting a Lancet

To insert a lancet:

1. Remove the orange lancet device cover from the Dario Housing, as shown in the images below.



2. Remove the used lancet, if one exists, by grasping and pulling it straight out of its housing.



To avoid accidental injury, dispose of used lancet in accordance with the instructions provided by your health care provider for biohazardous waste.

3. Remove a new lancet from the packaging.



Do not use the lancet if the cap is missing or damaged.

4. Insert a sterile lancet as far as it will go and ensure it is firmly set.



- Only use a lancet authorized by LabStyle Innovation Ltd. (30G)
- If the lancet does not set into place or the lancet holder does not hold the lancet properly, insert another sterile lancet
- If the problem persists, you may need to replace your Dario Housing. **Contact Customer Support**, for contact information please see page 98.

5. Carefully twist off the lancet cap, as shown in the image below. Ensure that you do not bend the lancet while removing the cap.



Setting Lancet Depth (Optional)

You can adjust the puncture depth of the lancet by moving the lever between the numbers 1 and 4 (printed on the dial). 1 is the shallowest depth and 4 is the deepest. It is recommended that you start with the lancet depth set to 1 and adjust the level upwards if necessary.

To set lancet depth:

1. Move the lancing device lever so that the line on the lever is aligned with the correct depth number, as shown in the image below.



2. Replace the lancing device cover by sliding it into place until it clicks. Be careful not to touch or bend the lancet.

The Dario App

The Dario App is available on the App Store for iPhone and on Google Play for Android devices. To ensure maintenance of service, please review the list of compatible devices in **Compatible Platforms with the Dario Blood Glucose Monitoring System supported** on page 21 or on:

www.mydario.com/support/getting-started

for the latest operating system and mobile device compatibility.

Your Dario Glucose Meter is operated from the Dario App displayed on the screen of your smart

mobile device. The mobile device displays and stores information received from the glucose meter. The Dario App also provides a step-by-step quick guide for operating the system.

You can use the Dario App to conveniently store and manage your diabetes information, review trends, or share information with your health care provider.

Downloading and Installing the Dario App

You can download and install the Dario App to your smart mobile device using the **Apple App Store®** or **Google Play**. The Dario App downloads and installs just like any other app on your smart mobile device. The Dario App is free of charge and is customized for your smart mobile device.



Downloading the Dario App from the Apple App Store or Google Play requires data connectivity.



For accurate use of the Dario System with the app, the time and date must be correctly set on your smart mobile device. For instructions on setting the time and date, refer to the documentation for your device.

To download the Dario App:

1. Turn on your smart mobile device and launch the Apple App Store (for iPhones) or Google Play (for Android). Alternatively, for quicker access, you can use an app that scans QR codes to scan the following code:



2. Search for the Dario App by LabStyle Innovation Ltd.
3. Install the App.
4. Follow the registration flow as shown on the App.
The application downloads. Once installed, the orange Dario icon appears on your smart mobile device home screen, as shown in the image below.




The Dario App needs to be installed once on each of your smart mobile devices.



Updates may be downloaded periodically from the App Store or Google Play.

Creating Your Account and Personalizing Your Settings

To setup your Dario App:

1. After installing the Dario App on your mobile device, press the orange **Dario icon** . The icon appears on your smart mobile device's Home screen.
2. Select **Sign Up** to register as a Dario user. Dario App registration requires email and Password to create your account.
3. Continue by following the instructions on the screen to register. You can register by entering your email address and password; or, alternatively, you can connect with Facebook.



Your password must have at least seven (7) characters and include an upper-case letter, a lower-case letter, and a number.




Your email address and password will be used to log in to the Dario App and the Dario web portal.




If you already have a Dario account, you can skip the next steps.

4. Press **Create Account** when done.


The First Entry screen appears, as shown in the image below.

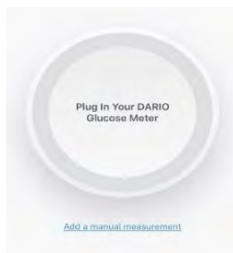

You can edit your email address by tapping the **Menu icon**  and selecting **MY PROFILE > ABOUT ME**. You can set the Dario App language and region by selecting **MY PROFILE > ACCOUNT SETTINGS**.

Setting Your Personal and Diabetes Profiles



To set your personal profile:

1. Tap the **Menu icon**  in the upper left corner and select **MY PROFILE > ABOUT ME**.
2. Tap **Edit** in the upper right corner to edit your name, email, phone number, condition, sex, date of birth, height, weight and other conditions. You can also upload a photo from your device by tapping **PHOTO icon**.
3. Tap on **Save** button to save the updated details.

To set your diabetes profile:

1. Tap the **Menu icon**  in the upper left corner and select **MY PROFILE > DIABETES PROFILE** to edit

diabetes type, date of diagnosis, insulin type, bolus medication, basal medication, oral medications, HbA1c details and caregiver information.

2. Tap the **Menu icon**  in the upper left corner and select **MY PROFILE > TARGET RANGES** to edit blood glucose thresholds, target weight and goals.
3. Tap the **Menu icon**  in the upper left corner and select **MY PROFILE > ACCOUNT SETTINGS** to select your preferred blood glucose and carbs measurement unit.
4. Tap on **Save button to save the updated details.**



You can complete the Walkthrough at any time by selecting **HELP&INFO > VIDEO GUIDES.**

CHAPTER 3

CHECKING YOUR DARIO GLUCOSE METER

This chapter provides information on checking the accuracy of your meter with a control solution.

To ensure that your results are always accurate, you need to check the performance of your meter in the following cases:

- If you drop the meter
- Whenever your result is inconsistent with how you feel
- If you have repeated a test and the blood glucose result is still lower or higher than expected
- Whenever you want to check the performance of the meter and a test strip.

Dario Control Solution

The Dario Control Solution confirms that the meter and test strips are performing as expected.

Shake the control solution container thoroughly before use. Squeeze out a drop and place it on the tip of the container cap. Hold the meter to move the yellow testing window of the test strip to touch the drop.

Compare the results on the meter with the values displayed on the test strip cartridge.

The Dario Control Solution package can be ordered separately as indicated in page 24. The Dario Control Solution package comes with two bottles of control solution (Level 1 (M), Level 2 (H)).

Control Solution Precautions

Note the following about the control solution:

- Avoid contact with your eyes. Contact your health care provider immediately if solution is swallowed, injected, or comes in contact with your eye.
- The dye in the control solution may stain skin, clothing, or surfaces.
- Store the control solution in an area where the temperature is between 46°F-86°F (8°C-30°C). Keep away from direct sunlight, fluorescent light, and heat.
- Do not freeze.
- Once you open a new bottle of control solution, mark the date on the bottle.
- Ensure that your control solution has not expired. Check the expiration date on the bottle.
- Ensure that your control solution was not opened more than three (3) months ago.
- Keep the control solution out of reach of children.
- Shake the control solution well before using.
- After each use, wipe the container tip clean and close the container tightly.
- No reconstitution or dilution of the control solution is necessary.

Control Solution Test Principle

The Dario Control Solution contains a known amount of glucose that reacts with chemicals on the Dario Test Strip.

Shake the control solution container thoroughly before use. Squeeze out a drop and place it on the tip of the container cap. Hold the meter to move the yellow testing window of the test strip to touch the drop.

The glucose in the control solution reacts with chemicals in the test strip.

The result displayed on the meter should fall within the expected range printed on the test strip cartridge label. You only need to obtain a single result within the expected range to verify system performance. A result outside the range printed on the test strip cartridge label may indicate the system is not performing correctly.



The control solution is not for use as a cleaning solution.

Chemical Composition

Dario Control Solution is an aqueous solution that contains d-glucose as the reactive ingredient in the approximate concentration.

The remaining chemical components of the Dario Control Solution are:

- Buffer 75mM
- Polyvinylpyrrolidone 5%
- Polyethylene glycol 5%
- SDS 0.1%
- Preservative 0.1%

Checking Your Glucose Meter with Dario Control Solutions

To perform a control solution test, you must perform the following three procedures:

- Removing the Glucose Meter from the Housing
- Removing a Test Strip from the Cartridge
- Performing a Control Solution Test

Removing the Glucose Meter from the Housing

Before performing a control solution test or testing your blood, remove the glucose meter from the housing. You should always store the glucose meter in the housing when you are not using it.



When not in use, store the meter in the Dario Housing. Do not leave your meter connected to your smart mobile device with the screen in ON mode for more than two (2) minutes, as the meter can get warm which can affect the accuracy of the reading.

To remove the meter from the housing:

1. Hold the housing so that the ridged orange panel is close to you and the end with the lancing device is facing away from you.
2. Slide your thumb back on the ridged orange panel in the direction indicated by the arrow, as shown in the image below. The meter will pop up.



3. Grasp the meter with two fingers and remove it from the housing.



Do not plug the Dario Glucose Meter into any device other than a compatible smart mobile device port. For a list of compatible devices, refer to -

www.mydario.com/support/getting-started/

Removing a Test Strip from the Cartridge

You should have a test strip cartridge already inserted in your Dario Housing. If this is not the case, refer to **Inserting the Test Strip Cartridge** on page 25.

To remove a test strip from the cartridge:

1. Remove the cartridge cover of the Dario Housing by pushing it outwards until it comes off (use the removal indication lines), as shown in the images below.



2. Snap open the test strips cartridge cap by pushing up the edge.

3. Remove a test strip from the cartridge, as shown in the image below.



4. Close the cartridge cap immediately and return the white cartridge cover to the closed position by pushing it into place until it clicks.



You should use the test strip immediately after removing it from the cartridge. It is not recommended that you use a test strip that has been outside of the cartridge for more than three (3) minutes.



If the cover does not close properly, the cartridge may not have been inserted correctly. Ensure that you have correctly, returned the cartridge cover to the closed position as described in step 4.



- Do not touch the yellow testing site when handling the test strip.
- Do not use a damaged test strip.

Performing a Control Solution Test

To perform a control test:

1. Tap the Dario icon on your smart mobile device to launch the Dario App.

The **First Entry** screen appears, as shown in the image below.



2. Insert the meter into the port of your smart mobile device, as shown in the image below.

A "Connecting..." message appears in the Dario App and then the screen displays the following instruction: **INSERT A NEW STRIP.**





Verify that the meter is inserted as far as it can go into the port of the smart mobile device and no other device or cable is connected to the smart mobile device.


3. Hold the test strip with the gold contacts facing the meter.
4. Without bending the strip, insert it into the test strip port (marked in orange) of the Dario Glucose Meter as far as it will go.



Verify the test strip direction, as shown in the image below.



The Dario App screen instructs you to place a drop of blood on the test strip, as shown in the image below.

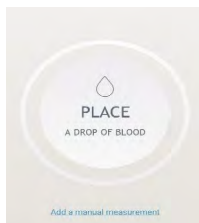


5. Shake the control solution container thoroughly before use. Squeeze out a drop and place it on the tip of the container cap. Hold the meter to move the yellow testing window of the test strip to touch the drop.




Important: Verify that you apply enough control solution to fill the window so that it turns red. Do not put control solution on top of the test strip.

When the meter detects the solution, the "Measuring..." message appears. The orange rotating in the circle indicates that the test is in progress, as shown in the image below.

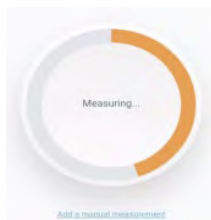

The glucose test result appears on the screen after six (6) seconds and is automatically saved to memory.



You can delete this control test from memory, if desired. Alternatively, the Dario System includes a function for specific control solution tests. Control solution tested values must be tested under the "Control solution test" function: **Menu icon** , selecting **HELP&INFO > CONTROL SOLUTION TEST** or

the results will be included in the blood glucose averaging functions of the device.

6. Compare the results on the meter with the values displayed on the test strip cartridge. If the results are:
 - Within the acceptable target values, then your meter is ready for use.
 - Outside of the acceptable target values, perform the test again with a new test strip. If the problem recurs, **Contact Customer Support**, for contact information please see page 98.
 -
7. When the test is done, remove the test strip and follow your health care provider's instructions for disposal.
8. Disconnect the meter from your smart mobile device.
9. Wipe the tip of the control solution container clean and tightly seal the container.



The Dario Control Solution target values printed on the test strip cartridge label are intended for the Dario Test Strip Cartridge only.



When measured by systems other than the Dario System, the glucose concentrations in these solutions may differ.




Applying too much or too little control solution to the test strip may cause out-of-range results.



Do not use your Dario System to test your blood if your control solution result falls outside of the acceptable target values.



The Dario Blood Glucose Monitoring System includes a function for specific control solution tests. Control solution tested values must be tested under the "Control solution test" function: **Menu icon** , selecting **HELP&INFO > CONTROL SOLUTION TEST** if not done the results will be included in the blood glucose averaging functions of the device.

10. Place the meter back into the housing by holding the meter at an angle with the part that connects to the smart mobile device facing away from the ridged orange panel. Then, push the meter into the housing at an angle and push downwards until the meter clicks into place.



CHAPTER 4

TESTING YOUR BLOOD GLUCOSE LEVEL

The Dario Glucose Meter measures the amount of glucose in your blood. Once you place a small drop of blood from your fingertip onto the test strip, the result will display in six (6) seconds on the screen in the Dario App.

Carefully read and follow the instructions provided in this chapter and in the package inserts for the test strips and control solutions. It is very important to follow the instructions to prevent an incorrect result or improper treatment.



Do not put any fluids into the test strip port.

Once you start the Dario App, follow the prompts on the screen.

Blood Glucose Testing Precautions



The meter, test strips, and control solutions are only intended for use outside the body (in vitro). Do not eat the test strips. Do not swallow or inject control solutions or use control solutions for any purpose other than testing your blood glucose.



FDA Public Health Notification: “Use of Fingertick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens: Initial Communication” (2010).

You can find this document at www.fda.gov.



CDC Clinical Reminder: “Use of Fingertick Devices on More than One Person Poses Risk

for Transmitting Bloodborne Pathogens” (2010). You can find this document at www.cdc.gov.



The meter and lancing device are for single-patient use only. Do not share them with anyone including other family members. Do not use on multiple patients.



It is advised that you wash your hands thoroughly with soap and warm water before and after handling the meter, lancing device, or test strips.

Incorrect test results may result in the wrong therapy recommendation and may lead to serious adverse health conditions. Therefore, follow the proceeding instructions carefully and read the items listed in **Test Strip Precautions** below.

Test Strip Precautions

- Only use Dario Test Strips. Using other test strips with the Dario Glucose Meter can produce inaccurate results.
- Inspect each test strip cartridge before opening it for the first time. If you see any damage to the cartridge lid, or if anything prevents the lid from closing properly, do not use the test strips.
- Run a control test every time you open a new test strip cartridge. For additional information, refer to **Checking Your Glucose Meter with Dario Control Solutions** on page 36.

- Store unused test strips in their original cartridge. Do not remove test strips from the test strip cartridge and put them into another container.
- Close the cartridge tightly right after you remove a test strip. This helps to keep the test strips dry.
- Check the “Use by” date on the test strip cartridge. Do not use the test strips after that date.
- Write down the date the test strip cartridge was first opened. Check this date before you use a test strip from this cartridge. If the current date exceeds the shelf life, do not use the test strips from this cartridge. Refer to the Instructions for Use information on the test strip cartridge packaging.
- Do not move tests strips from one cartridge to another.
- Use the test strip immediately after removing it from the cartridge.
- Use the test strips as indicated in this guide.
- Do not store your test strips in temperatures below 36°F (2°C) or above 90°F (32°C).
- Store your test strips in an area where the relative humidity is between 10-90%.
- Do not place the test strips in a freezer.
- Do not expose your test strips to direct sunlight or moisture.
- Do not apply blood or control solution to the test strip before inserting it into the glucose meter.
- Do not reuse test strips. Once control solution or blood is placed on a test strip, discard the strip. If you need to retest, use a new test strip.

- Do not bend, cut, or alter the test strips.
- Do not get dirt, food, or other substances on the test strip.
- It is recommended that you use the test strip immediately after removing it from the cartridge. It is not recommended that you use a test strip that has been outside of the cartridge for more than three (3) minutes.
- Do not touch the yellow testing site when handling the test strip.
- Do not use a damaged test strip.
- Make sure that the gold contact area of the test strip is clean from dirt, blood, moisture, and any other material. An unclean test strip will affect the test result and can cause an inaccurate blood glucose level result.

Factors That Interfere with Blood Glucose Testing

Certain substances can interfere with the blood glucose system and cause inaccurate results. For more information, refer to the test strip package insert. If you are taking medication or dietary supplements, consult your health care provider. Refer to **System Limitations** on page 8 for full list of interfering substances.



Certain health conditions, such as serious illness and dehydration, can cause false test results in glucose meters. If you are very sick or think you are dehydrated, or are urinating frequently, call your health care provider immediately.



You should not use this system to test your blood glucose if you are critically ill (examples

include: severe hypotension or shock, hyperglycemic-hyperosmolar state, hypoxia, and diabetic ketoacidosis).



Unwashed hands or wet hands could affect your results. Alcohol can affect test results.

Interpreting the Results

If the displayed test result is consistent with how you feel, continue as instructed by your health care provider.



If the test result is inconsistent with how you feel, check your Dario System by running a control solution check, as described in

Performing a Control Solution Test on page 39. Repeat the blood glucose test afterward. If your blood glucose result is still inconsistent with how you feel and you have followed the instructions provided in this chapter, follow your health care provider's instructions.



Do not change your treatment based on a single result that is inconsistent with how you feel or if you believe that your test result is incorrect. The blood glucose reading displayed on your smart mobile device by Dario System should not be the only information you use to make decisions about your health. Always consult your health care provider to interpret and understand the measurement results.

Measurement Range

Your meter measures blood glucose results in the range of 20 mg/dL to 600 mg/dL (1.1 - 33.3 mmol/L). Results outside of this range should be reviewed according to the following information.

- If your test result is lower than 20 mg/dL (1.1 mmol/L), “LOW! Measure with a new strip, if recurs contact your physician immediately” displays instead of a result.



LOW may indicate that your blood glucose is very low (possibly severe hypoglycemia). Follow your health care provider's instructions immediately and repeat the test.

- If the result is higher than 600 mg/dL (33.3 mmol/L), “HIGH! Measure with a new strip, if recurs contact your physician immediately” displays instead of a result.
- If your blood glucose test results in a **LOW!** or **HIGH!** message, repeat the blood glucose test following all instructions in **Step Five: Testing Your Blood Glucose** on page 55.
- If your blood glucose test results in a LOW! or HIGH! message a second time, follow the protocol established by your health care provider or contact your health care provider immediately.
- Note that many factors determine whether your blood glucose reading is outside the range of 20 mg/dL to 600 mg/dL (1.1 - 33.3 mmol/L), including individual differences and the time since your last meal.

The American Diabetes Association suggests that the following expected glucose values for individuals without diabetes:

Before eating (FPG) < 100 mg/dL (5.6 mmol/L)

Two hours after meals (OGTT) < 140 mg/dL (7.8 mmol/L)

Reference: American Diabetes Association (2017). Standards of Medical Care in Diabetes – 2017. Diabetes Care, 40 (Supplement 1): S1-S132.

Factors That May Lead to Inaccurate Results

Certain medications, food supplements, medical treatments, and health conditions can interfere with the blood glucose system and cause inaccurate results.

Incorrect testing technique or improper storage or handling of the Dario System may also lead to inaccurate test results which are inconsistent with how you feel. The following conditions could lead to incorrect results:

- Your hands may not have been washed thoroughly.
- Your glucose meter test strip port may be contaminated.
- Your test strip has been bent.
- The test strip is expired. Check the Use By date.
- The length of time that has passed since first opening the test strip package exceeds the shelf life.
- Test strips were stored incorrectly.

- Testing was done in a humid environment (for humidity specifications, refer to **General Precautions** on page 15).
- Your glucose meter has been dropped



If your meter has been dropped and you are receiving error messages or implausible results, **Contact Customer Support**, for contact information please see page 98. Do not rely on the blood glucose measurement alone in making health care decisions.

Step One: Inserting a New Lancet into the Lancing Device



To prevent infection, always use a new, sterile lancet in your lancing device each time you test your blood glucose.

Insert a new lancet into the lancing device, as described in **Inserting a Lancet** on page 26.

If necessary, set the depth of the lancing device. For more information, refer to **Setting Lancet Depth (Optional)** on page 28.

Return the lancing device (orange) cover by sliding it into place until it clicks. Be careful not to touch or bend the lancet.

Step Two: Opening the Dario App

Before connecting the Dario Glucose Meter into your smart mobile device, open the Dario App by tapping the **Dario icon** on your smart mobile device Home screen.



Step Three (relevant to the US market only): Inserting the Smart Mobile Device into the Disposable Cover

Before starting the testing procedure, you should cover your phone with a disposable cover.



To avoid the risk of transmitting bloodborne pathogens, it is recommended that you use a disposable sleeve to cover your smart mobile device while taking the blood glucose test.

The disposable cover protects the mobile device from direct contact with blood during the glucose measurement.

Insert your smart mobile device into the disposable cover so that the opening at the bottom of the sleeve fits over the Lightning or USB-C port, as shown in the image below.



Step Four: Removing the Glucose Meter from the Housing

Before starting the testing procedure, remove the meter from the housing, as described in **Removing the Glucose Meter from the Housing** on page 37. You should always store the meter in the housing when you are not using it.

Place the glucose meter connector into the smart mobile device port at the bottom of the disposable cover. Verify that the meter is inserted as far as it goes into the port of the smart mobile device and no other device or cable is connected to the smart mobile device.

A “Connecting...” message appears in the Dario App, and then the screen displays the following instruction: INSERT A NEW STRIP.



The Dario Meter should be inserted with logo facing up.



The screen notification

Step Five: Testing Your Blood Glucose

You are now ready to test your blood glucose.

Before beginning the test, ensure you understand the information provided in **Testing Information and Guidelines**.

Testing your blood glucose consists of the following two procedures:

- **Removing a Test Strip from the Cartridge**
- **Performing a Glucose Test**

Testing Information and Guidelines

- To perform the test, the meter requires only a small drop of blood about the size of a sugar grain (0.3 μ L). For best results, select a puncture site along the side of the fingertip.
- While applying blood to the test strip, verify that you apply enough blood to fill the test strip window so that it turns completely red.
- Testing blood from sites other than the fingertip may generate inaccurate results.
- If you have not applied enough blood to the test strip, an alert displays that informs you that the sample size is too small and you should try again

with a new strip. Blood glucose results are automatically saved to mobile App memory.

- If a test strip error occurs (e.g. "used strip"), perform another test with a new test strip.
- If an error message or symbol appears on the screen, refer to **CHAPTER 7 TROUBLESHOOTING** on page **78** for additional information.
- Unwashed hands could affect your test results.
- Alcohol can affect test results.
- When not in use, store the meter in the Dario Housing. Do not leave your meter connected to your smart mobile device with the screen in ON mode for more than two (2) minutes, as it can get warm and can affect the accuracy of the test.



To prevent malfunction of the meter, keep the test strip port free of blood, moisture, dirt, or dust. Always store the glucose meter in the Dario Housing when not in use.



In case of a meter error, you may need to remove and insert the meter to allow the meter to reconnect to the smart mobile device.



In case of a test strip error you may need to replace the test strip and test with a new test strip.

Removing a Test Strip from the Cartridge



You should have a test strip cartridge already inserted in your Dario Housing. If this is not the case, refer to **Inserting the Test Strip Cartridge** on page **25**.

To remove a test strip from the cartridge:

1. Wash your hands with warm, soapy water and dry thoroughly. If you use alcohol wipes to clean your fingers, ensure the puncture area is completely dry before lancing your finger.
2. Remove the cartridge cover of the Dario Housing by pushing it outwards until it comes off (use the removal indication lines), as shown in the following image.



3. Open the lid by pushing up the edge, as shown in the image below.



4. Remove a test strip from the cartridge, as shown in the image below.



5. Close both the lid and cartridge cover.



If the cover does not close properly, you may not have inserted the cartridge correctly.

6. Hold the test strip with the gold contacts facing upward.


7. Without bending the strip, insert it into the test strip port of the Dario Glucose Meter as far as it will go.



Verify the test strip direction, as shown in the image below.



The Dario App instructs you to place a drop of blood on the test strip, as shown in the image below.

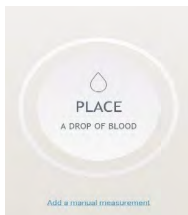

Performing a Glucose Test



If you receive a phone call while the glucose measurement is in progress, Dario will stop the test and the results will not be saved. To avoid interruption of the glucose test, you can set your phone to airplane mode before beginning this procedure. For more information, refer to your mobile device's instructions.

1. Set the lancing device for activation by sliding it with two fingers as far as it will go, as shown in the following image.



2. Place your fingertip over the hole on the lancing device and press the orange release button with your thumb to prick your finger, as shown in the images below.



3. Apply a drop of blood to the tip of the yellow window of the test strip, as shown in the images below. Capillary action will pull the blood into the strip.



Important: Continue applying blood to the test strip until the window turns red, which indicates that enough blood has been collected to take a reading. Do not put blood on top of the test strip.



When the meter detects that blood has been applied, the “Measuring...” message appears in the Dario App. The border of the circle gradually fills with orange, indicating that the test is in progress.



4. Remove your finger from the strip once the above message appears.



Do not add blood to the test strip once you have removed your finger because you may receive an inaccurate result.

5. The test result appears on the screen after approximately six (6) seconds.

The glucose test result is automatically saved to memory. If a result does not display, you can view it in your Logbook in the Dario App.



In case an error message appears, you may need to start the measurement process again. If error recurs, refer to **CHAPTER 7 TROUBLESHOOTING** on page 78 section for further information.



If you receive a phone call while a measurement is in progress, the test will stop and the results will not be saved. The following message will display on the result screen: “The Test Strip Is Used” and you will have to start the test over with a new test strip. To avoid an interruption of your glucose test, you may want to set your smart mobile device to airplane mode prior to starting your test.

6. You may continue to complete the information about the glucose measurement that displays on the Dario App screen (e.g., measurement setting, carb consumption, exercise, insulin intake).
7. When the test is done, remove the test strip and disposable cover and dispose of them in accordance with the instructions provided by your health care provider for biohazardous waste.
8. Remove the lancet from the lancing device, as described in step 2 in **Inserting a Lancet** on page 26 and dispose of the used lancet in accordance with the instructions provided by your health care provider for biohazardous waste.

When the test strip is removed from the glucose meter, the following message appears in the Dario App: "Please insert new test strip."

Step Six: Returning the Meter to the Dario Housing

To return the meter to the Dario Housing:

1. Disconnect the meter from your smart mobile device.
2. Hold the meter at an angle with the meter plug facing away from the ridged orange panel.
3. Push the meter into the housing at an angle and then push downwards until the meter clicks into place.

Test Strip Limitations

If you are taking acetaminophen containing drugs (e.g. Tylenol) or Vitamin C (ascorbic acid) you may get inaccurate results with this system. If you have a disease or condition in which uric acid levels in your blood may be elevated (> 11 mg/dL), such as gout,

you may get inaccurate results with this system. If you have very high levels of triglyceride (> 810 mg/dL), you may get inaccurate results with this system. This system should not be used when undergoing xylose absorption tests.

Packed cell volume: Dario BGMS Test Strip suitable range is 20~60%.

CHAPTER 5

CLEANING AND MAINTENANCE

This chapter provides information for cleaning and disinfecting for your Dario Glucose Monitoring System.

The Dario System should be cleaned and disinfected once a week. The Dario System should be properly maintained for safe and accurate use. The meter should be properly cleaned and disinfected to prevent the spread of infectious diseases.

Cleaning and Disinfection

The outer surfaces should be cleaned when you notice that your Dario is visibly dirty. It is advised to clean and disinfect your Dario System once a week to avoid transmitting bloodborne pathogens.

If someone else is operating the meter to assist you with testing, he or she should clean and disinfect the meter, housing, orange lancet cover and white cartridge cover prior to use.

“Cleaning and disinfection” is a two steps process and should be done once a week. Cleaning and disinfection refer to different actions. Cleaning removes dirt and blood while disinfection kills the bloodborne pathogens. Before disinfecting your system, first clean the meter, housing, orange lancet cover, and white cartridge cover.



Failure to properly clean your device prior to disinfection can damage your device and/or transmit blood borne pathogens.

Wash your hands thoroughly with soap and water before and after handling the meter, housing,

orange lancet cover and white cartridge cover, or test strips.

Step 1: Cleaning

Clean the meter, housing, orange lancet cover, and white cartridge cover using a disinfection wipe:

- In the USA - use a Super Sani-Cloth
- In Canada – use Germidice³

These wipes have been proven safe for use with the Dario meter, housing, orange lancet cover, and white cartridge cover. Super Sani-Cloth and Germidice³ are available online through [amazon.com](https://www.amazon.com) and [officedepot.com](https://www.officedepot.com). Use the germicidal disposable wipe to remove visible dirt, blood, or other substances. Before cleaning, remove the meter from its housing. The meter must not be connected to the mobile device while cleaning.

Clean the meter, housing, orange lancet cover, and white cartridge cover by wiping the outside surfaces with a germicidal disposable wipe to remove all visible dirt. Remove the white cartridge cover and wipe the cartridge cap as well. Before you begin, squeeze out any excess liquid from the wipe so that it is damp but not dripping wet. Take extreme care not to get liquid on the test strip port.

Step 2: Disinfection

Prior to disinfection ensure you have properly cleaned your device as recommended in **Step 1: Cleaning**. Disinfect the meter, housing, orange lancet cover, and white cartridge cover using a clean Super Sani-Cloth or Germidice³ wipe to kill the bloodborne pathogens that may spread infection.

Using a second clean wipe:

- Disinfect the meter on all four (4) surfaces by wiping up and down at least two (2) times to each direction, as shown in the images on the following page. Do not allow any moisture to get into the test strip port.
- Disinfect the housing, orange lancet cover, and white cartridge cover by wiping the outer surfaces. Ensure that you wipe all parts including the housing, the white cartridge cover, and the orange lancet cover up and down at least two (2) times in each direction as demonstrated in the images on the following page.
- Remove the white cartridge cover and disinfect the cartridge cap by wiping it up and down thoroughly at least two (2) times in each direction as demonstrated in the images on the following page.
- It is very important to let the liquid from the wipe sit on the outer surface for at least two (2) minutes. Be extremely careful to avoid getting liquid onto the test strip port.

The following images demonstrate how to wipe to clean and disinfect your Dario System.





To ensure effective disinfection, let the solution sit on the meter, housing, orange lancet cover, and white cartridge cover for at least two (2) minutes. Allow the meter, housing, orange lancet cover, white cartridge cover and cartridge cap to completely dry before use. Wipe off any remaining residues with a clean, soft, and dry paper cloth.

Your Dario System has an expected life of three (3) years. It should withstand the cleaning and disinfection recommended (156 cycles) in this user guide.

It is recommended that you clean and disinfect the entire system at least once per week and whenever you notice blood, dirt, or visible residue.

If you notice changes in appearance (such as color fading, cracking, discoloration or display malfunctions) or you are obtaining results that are not accurate, stop using the meter or lancing device and call customer support:

- USA: toll-free Monday to Friday, 9:00AM – 9:00PM EST at 1-800-895-5921 or www.mydario.com
- Canada: toll-free at 1-800-461-0991 or www.mydario.ca

Please contact your health care provider for questions outside of customer support hours.

The cleaning and disinfection performance testing on the Dario System was done using the Super Sani-Cloth (in Canada use Germidice³) with EPA registration number 9480-4 (Active ingredient: Quaternary ammonium chlorides and isopropanol).

“Cleaning and disinfection” is important to prevent the spread of disease. If you are interested in additional information about the importance of disinfection, refer to the following articles:

- “FDA Public Health Notification: Use of Fingerstick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens: Initial Communication” (2010). You can find this document at www.fda.gov
- “CDC Clinical Reminder: Use of Fingerstick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens” (2010). You can find this document at www.cdc.gov



Do not:

- Allow any moisture to get into the test strip port.
- Spray any cleaning solution directly onto the glucose meter or the lancing device.
- Submerge any part of the Dario System in liquid.
- Attempt to clean any internal parts.
- Attempt to dry any parts using a blow dryer.

Please observe the following cautions when cleaning the Dario System to avoid damaging the device:



If the device becomes wet, wipe off all moisture and allow sufficient time for the device to dry before using.



To avoid the risk of transmitting bloodborne pathogens, it is recommended that you use a disposable sleeve to cover your smart mobile device while taking the blood glucose test. For more information about using the disposable cover, refer to instruction on how to use **Disposable Covers** on page 53.



If you notice signs of deterioration such as clouding, cracking, crazing, or if the meter stops functioning as expected, stop using the meter and **Contact Customer Support**, for contact information please see page 98. Please contact your health care provider outside of the customer support hours.

CHAPTER 6

TEST STRIPS

Important: Dario test strips are designed for use with the Dario®, Dario® LC and Dario® USB-C Blood Glucose Monitoring Systems. Please read these instructions for use before using this product.

Product Description and The Principle of The Use

Dario Blood Glucose Test Strip is designed with an electrode that measures glucose levels. Glucose in the blood sample mixes with reagent on the test strip that causes a small electric current. The amount of current that is created depends on how much glucose is in the blood.

The Dario, Dario LC and Dario USB-C meters measure the current that is created and convert the measurement to the amount of glucose that is in the blood. The blood glucose result is displayed on Dario App screen.

By touching a drop of blood to the tip of Dario Blood Glucose Test Strip, the strip's reaction chamber automatically draws the blood into the strip through capillary action. When the chamber is full the Dario, Dario LC and Dario USB-C meters start to measure the blood glucose level. It is a simple and practical system for the daily monitoring of your blood glucose level.

STORAGE AND HANDLING

1. Store the test strips at room temperature between 36°F and 90°F (2°C and 32°C). Do not freeze. Frozen and thawed reagents may cause incorrect glucose results.

2. Keep the test strip port free of dust.
3. The test strip is sensitive to humidity, keep it in 10%-90% Relative Humidity and cool environment.
4. Do not store in direct sunlight.
5. After pulling out the test strip from its cartridge, seal the cap of the cartridge tightly to protect test strips.
6. Do not transfer test strips from one cartridge to another.

The Dario Test Strip is structured as follows:



Yellow Window

Apply a drop of blood here.

Before Applying Blood –

This window should be completely yellow.

- **After Applying Blood –** This window should be completely filled with the blood drop.

Electrodes

Insert a strip with gold-colored bars up and toward the meter.

Reagent Composition (per strip)

Glucose oxidase (<i>Aspergillus sp</i>)	1.5 unit
Potassium ferricyanide	0.093mg
Binder	0.052mg
Stabilizer	0.031mg

Precautions

- Use only fresh capillary whole blood by finger prick for blood glucose testing.
- A test strip should be used according to the specified intended use.

- A test strip is for single use only. Do not reuse
- Dario Blood Glucose Test strips should be used with Dario/Dario LC/Dario USB-C meters only.
- Insert a test strip into the test strip port in the glucose meter with gold-colored bars facing up and toward the meter.
- A test strip is sensitive to humidity. Therefore, you should keep a test strip in the specified cartridge, and after pulling out the test strip from its cartridge, close the test strip cartridge cap immediately.
- After removing a test strip from its cartridge, you should use it within 3 minutes.
- Do not use a test strip expired 1 month from opening. Discard the cartridge 1 month after opening.
- Insert a test strip in the meter port gently until it goes no further.
- The blood sample for a test strip must be at least 0.3µl in volume. If there is not enough blood sample an inaccurate test may result.
- Do not apply blood to any site except the yellow window of a test strip.
- Do not touch the yellow window of a test strip.
- Do not bend, cut or modify a test strip.
- Discard the used test strip as a biohazard.

System Limitations

Refer to **System Limitations** on page 8 for full list of interfering substances.

Performing a Test

Refer to Chapter 4 **TESTING YOUR BLOOD GLUCOSE LEVEL** on page 45 for full list of interfering substances.

Understanding Your Testing Result

Normal blood glucose result

- The American Diabetes Association suggests that the following expected glucose values for individuals without diabetes:
 - Before eating (FPG) < 100 mg/dL (5.5 mmol/L)
 - Two hours after meals (OGTT) < 140 mg/dL (7.8 mmol/L).
- Consult with your healthcare professional for the blood glucose range that is appropriate for you.

Test Result Range

The Dario, Dario LC and Dario USB-C meters read blood glucose results between 20-600 mg/dL (1.1-33.3 mmol/L).

Unexpected Results

High or low blood glucose results can indicate potentially serious medical conditions. In case of an unexpected result, you should repeat the test immediately using a new test strip. If your reading is still unexpected or the reading is not consistent with how you feel, you should treat as prescribed by your healthcare professional and/or contact your healthcare professional immediately.

- If “High! Measure with a new strip, if recurs contact your physician immediately” is displayed, your blood glucose result may be higher than 600

mg/dL (33.3 mmol/L). You may have high blood glucose.

- If “Low! Measure with a new strip, if recurs contact your physician immediately” is displayed, your blood glucose result may be lower than 20 mg/dL (1.1 mmol/L). You may have low blood glucose.

Control of The Unexpected Results

If your blood glucose result seems unusually high, low, or inconsistent with your previous results or glucose trends and does not reflect the way you feel, check the following:

- Repeat the test with a new strip.
- Run a control solution test with Dario Glucose Control Solution.
- If the control solution test result is within the acceptable range, review proper testing procedure and repeat your blood glucose test with a new test strip. If your blood glucose value is still inconsistent with your previous results, glucose trends, or how you feel, please contact your healthcare professional. Follow the advice of your healthcare professional before you change your therapy.

Causes of Unexpected Results

- Was the blood sample applied to the test strip within 3 minutes of removing it from the cartridge?
- Was the size of the blood sample sufficient to fill the yellow window?
- Was the test strip cartridge cap tightly sealed?

- Was the test strip used before the expiration date?
- Were the test strips stored at extreme temperatures such as in the car during very cold or hot weather?
- Were the test strips stored in areas of high humidity such as the kitchen or the bathroom?

Checking by a Control Solution

Control solution test

Control solution check ensures that you are doing a test correctly and that your system is working properly. Make sure you use the proper control solution for the test strips you have: Dario® Blood Glucose Test Strips require Dario® Glucose Control Solution.

When should I run a control solution test?

- If you leave the cartridge cap open.
- Before using your meter for the first time.
- When you open a new cartridge of test strips.
- If you drop the meter.
- Whenever your result does not agree with the way you feel.
- If you have repeated a test, and the blood glucose result is still lower or higher than expected.
- When you want to check the performance of the meter and test strip



Do not use Dario Control Solution expired 3 months from opening. Therefore, please check the date of opening for the accurate review of the meter and the test strip.



Test control solution at 64-86°F (18-30°C) environment.

Control Solution Procedure

To conduct a control solution test, refer to **Performing a Control Solution Test** on page 39.

Understanding Your Control Solution Test

- If your glucose control solution test result is within the acceptable range, you can feel confident that your test strips and meter are working correctly.
- If your glucose control solution test result is outside of the acceptable range, your system may not be working properly.

Check the following:

- Did you use control solution or test strips that were past their expiration date?
- Did you leave the cap off the cartridge of test strips or control solution?
- Did you follow the testing steps exactly?
- Did you use the proper control solution for the test strips you used?

Repeat the glucose control solution test. If the result still is not acceptable **Contact Customer Support**, for contact information please see page 98.



Disclaimer: While every precaution has been taken to ensure the diagnostic ability and accuracy of this product, the product is used outside of the control of the Manufacturer and Distributor and the result may

accordingly be affected by environmental factors and / or user error. A person who is the subject of the test should consult a doctor for further confirmation of the result.



The Manufacturers and Distributors of this product shall not be liable for any losses, liability, claims, costs or damages whether direct or indirect or consequential arising out of or related to an incorrect diagnosis, whether positive or negative, in the use of this product.

Bibliography

1. American Diabetes Association (2017). Standards of Medical Care in Diabetes – 2017. Diabetes Care, 40 (Supplement 1): S1-S132.
2. Stedman, TL. Stedman's Medical Dictionary, 27th Edition, 1999, p.2082.
3. Ellen T. Chen, James H. Nichols, Show-Hong Duh, Glen Hortin, MD:Diabetes Technology & Therapeutics, Performance Evaluation of Blood Glucose Monitoring Devices, Oct 2003, Vol. 5, No. 5: 749 -768.

CHAPTER 7


TROUBLESHOOTING

This chapter provides information about all the symbols and error messages that may display in the Dario App. To ensure safe and accurate use of Dario, pay attention to all symbols and messages that occur while using the system as they provide you with important information.

If you still have questions after reading this chapter, **Contact Customer Support**, for contact information please see page 98.

Please contact your health care provider for questions outside of the customer support hours.

An error message can appear in the Dario App as a message within the results area, as shown in the images below.

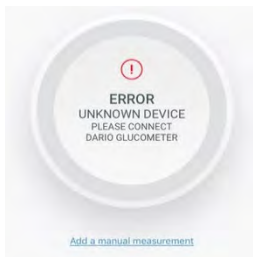

Troubleshooting

The following table lists some typical conditions that may occur with the Dario System.

Message Content	Possible Cause / Recommended Action
Insufficient blood detected, try again with a new strip	There is not enough blood on the test strip.
	Repeat the measurement with a new test strip.
The strip is used, please insert a new strip	A test strip was used more than once.
	Perform a new measurement with a new test strip.
Temperature out of range. Move to an area between 50°-95°F (10-35°C), wait 30 minutes and try again	The meter was left in an extreme temperature environment.
	Move to a cooler or warmer area, wait at least 30 minutes and repeat the measurement with a new test strip.
HIGH! Measure with a new strip, if recurs contact your physician Immediately	Glucose level is above operating range.
	Measurement result is above 600 mg/dL (33.3 mmol/L). Repeat the measurement with a new test strip. If this message recurs, contact your health care provider Immediately. If your measurement results are inconsistent with the way you feel, contact customer support.

Message Content	Possible Cause / Recommended Action
LOW! Measure with a new strip, if recurs contact your physician immediately	Glucose level is below operating range
	Measurement result is below 20 mg/dL (1.1 mmol/L). Repeat the measurement with a new test strip. If this message recurs, contact your health care provider Immediately. If your measurement results are inconsistent with the way you feel, contact customer support.
Dario device failure (0x00). If the problem occurs again, please contact Dario's customer support.	Communication failure.
	Try disconnecting and reconnecting the meter.
	Clean the lightning or USB-C connector with a dry cloth, if problem recurs, contact customer support.
Dario device failure (0x01). If the problem occurs again, please contact Dario's customer support.	Firmware software failure.
	Try disconnecting and reconnecting the meter.
	Clean the lightning or USB-C connector with a dry cloth, if problem recurs, contact customer support.
Dario device failure (0x02). If the problem occurs again, please contact Dario's customer support.	Voltage failure.
	Try disconnecting and reconnecting the meter.
	Clean the lightning or USB-C connector with a dry cloth, If problem recurs, contact customer support.

Message Content	Possible Cause / Recommended Action
Unknown device, please connect Dario.	Another device is connected to your smart mobile device's Lightning or USB-C port.
	If another device is connected to the Lightning or USB-C port, disconnect it and connect the Dario Glucose Meter. You can still use the Dario App if another device is connected. If you received this message while the Dario Glucose Meter is connected, disconnect and reconnect the meter. If this message still appears, contact customer support.
Hypo warning	A hypoglycemia event occurred.
	You received a measurement result lower than defined in the hypo warning section under Blood Glucose Thresholds. Contact your health care provider immediately.

Problem Solving

Test strip inserted but “Please insert a new strip” keeps appearing on the screen.

The test strip was removed before the reading was completed or test strip or meter is defective.

- Repeat the measurement using a new test strip
Check the meter with control solution, as
described in **Checking Your Glucose Meter with**

Dario Control Solutions on page 36). If the result is outside of the acceptable target values, **Contact Customer Support**, for contact information please see page 98. If the result is within the acceptable target values, test your glucose again with a new test strip.

You cannot pierce skin with lancet.

- The needle depth may be incorrect. Adjust the lancet depth lever to a different depth. For more information, refer to **Setting Lancet Depth (Optional)** on page 28.
- The lancet needle may be bent. Replace the lancet. For more information, refer to **Step One: Inserting a New Lancet into the Lancing Device** on page 26.
- The cartridge cover may be in the place of the lancing device cover. Please be careful to return each cover to the correct end.

The lancet does not fit correctly into the lancing device.

The lancet may be faulty or you may be using a lancet that is not approved for use with the Dario System. Replace the lancet with a different Dario Lancet. If the problem persists, you may need to replace the Dario Housing. Contact Customer Support, for contact information **please see page 98**.

Cannot remove the used needle from the lancing device.

Pull the needle carefully and straight out from lancet.

Replace the lancet with a different Dario Lancet. If the problem persists, you may need to replace the Dario Housing. Contact Customer Support, for contact information please see [page 98](#).

Lancing device does not operate correctly.

The lancing device spring or handle may be faulty. Ensure that you set the lancing device for activation by sliding it with two fingers until it clicks. Then, press the button to puncture your skin. For more information, refer to step 1 in performing a glucose test on [page 52](#)).

Lancet Depth Indicator lever is stuck.

The indicator lever may be faulty. Replace the Dario Housing. Contact Customer Support, for contact information please see [page 98](#).

Lancing device release button does not work properly.

The lancing device is faulty. Replace the Dario Housing. Contact Customer Support, for contact information please see [page 98](#).

“Strip is used and should be replaced” message appears in the Dario App.

This message appears if you receive an incoming call. It is recommended that you set your smart mobile device to airplane mode before performing a test.

I forgot my Dario password.

If you forget your password, tap on **FORGOT PASSWORD** on the **LOG IN screen**. You will receive an email with instructions for resetting your password.

My smart mobile device battery died.

If the battery of your smart mobile device is out of power, charge your device or use a different device. Download the Dario App onto that device and login to your account.

CHAPTER 8

TECHNICAL INFORMATION AND SPECIFICATIONS

This chapter provides technical information about your Dario System.

How the Glucose Meter Works

The Dario System determines the amount of glucose in fresh capillary blood using electrochemical biosensor technology. An electrical current is generated by the reaction of glucose with the reagent on the test strip (glucose oxidase). The strength of the current produced by the reaction depends on the amount of glucose in the blood sample. The glucose meter detects the current and converts it into a blood glucose reading.

Compatibility

Dario is compatible for use on selected smart mobile devices. For a complete list of compatible devices, please refer to:

www.mydario.com/support/getting-started/

Performance

User Performance/Accuracy*

System accuracy was evaluated by 350 fingertip blood glucose measurements, performed by 350 subjects using three test strips lots. Dario blood glucose results were then compared to the YSI® analyzer. Acceptance criteria in ISO15197:2015 are Ninety-five percent (95%) of the measured glucose values shall fall within either ± 15 mg/dl (0,83 mmol/l) of the averaged measured values of the reference measurement procedure at glucose concentrations <100 mg/dl (5.5 mmol/l) or within $\pm 15\%$ at glucose concentrations ≥ 100 mg/dl (5.5

mmol/l). Results were calculated across all data points and are summarized below:

Accuracy: Dario® LC BGMS (Dario with Lightning connector)					
For glucose concentrations < 100mg/dL (5.5 mmol/l)			For glucose concentrations ≥ 100mg/dL (5.5 mmol/l)		
Within ± 5 mg/dL (0.28 mmol/L)	Within ± 10 mg/dL (0.56 mmol/L)	Within ± 15 mg/dL (0.83 mmol/L)	Within ±5%	Within ±10%	Within ±15%
65/118 55.1%	105/118 89.0%	116/118 98.3%	130/232 56.0%	198/232 85.3%	226/232 97.4%

User Performance is performed in accordance with “Self-Monitoring Blood Glucose Test Systems for Over-the-Counter Use - Guidance for Industry and Food and Drug Administration Staff” - October 2016. The Dario® LC study showed the following results for all subjects’ samples: “96.3% within ±15% and 99.4% within ±20% of the medical laboratory values at the entire glucose concentrations range”.

Method comparison: Dario® LC BGMS (Dario with Lightning connector)						
For glucose concentrations < 75mg/dL (4.2 mmol/l)			For glucose concentrations ≥ 75mg/dL (4.2 mmol/l)			
Within ± 5 mg/dL (0.28 mmol/L)	Within ± 10 mg/dL (0.56 mmol/L)	Within ± 15 mg/dL (0.83 mmol/L)	Within ±5%	Within ±10%	Within ±15%	Within ±20%
23/38 (60.5%)	34/38 (89.5%)	38/38 (100%)	168/312 (53.8%)	261/312 (83.7%)	304/312 (97.4%)	311/312 (99.7%)

Precision*

The criteria are within 95% for standard deviation (STD) 3mg/dL (0.17 mmol/L) at the below 100mg/dL (5.55mmol/L), for coefficient of variation (CV) 4% at the above 100mg/dL (5.55mmol/L). The precision studies using control solution (Intermediate

Precision) and blood (Repeatability Precision) are shown below.

Intermediate Precision: Dario® LC BGMS					
Glucose Concentration range	30-50 mg/dL (1.7-2.8 mmol/L)	51-110 mg/dL (2.9-6.1 mmol/L)	111-150 mg/dL (6.2-8.3 mmol/L)	151- 250 mg/dL (8.4-13.9 mmol/L)	251-400 mg/dL (14-22.2 mmol/L)
Average	48.6 mg/dL (2.70 mmol/L)	85.6 mg/dL (4.76 mmol/L)	128.0 mg/dL (7.11 mmol/L)	203.7 mg/dL (11.32 mmol/L)	286.8 mg/dL (15.93 mmol/L)
Standard Deviation	0.13 mmol/L (2.3 mg/dL)	0.17 mmol/L (3.0mg/dL)	0.27 mmol/L (4.8 mg/dL)	0.43 mmol/L (7.8 mg/dL)	0.55 mmol/L (9.9 mg/dL)
Coefficient of Variation	4.7%	3.5%	3.8%	3.8%	3.4%
Max	55 mg/dL (3.06 mmol/L)	95 mg/dL (5.28 mmol/L)	139 mg/dL (7.72 mmol/L)	228 mg/dL (12.67 mmol/L)	322 mg/dL (17.89 mmol/L)
Min	140 mg/dL (2.22 mmol/L)	76 mg/dL (4.22 mmol/L)	114 mg/dL (6.33 mmol/L)	183 mg/dL (10.17 mmol/L)	266 mg/dL (14.78 mmol/L)
Max-Min	0.83 mmol/L (15 mg/dL)	1.06 mmol/L (19 mg/dL)	1.39 mmol/L (25 mg/dL)	2.50 mmol/L (45 mg/dL)	3.11 mmol/L (56 mg/dL)

Repeatability Precision: Dario® LC BGMS					
Glucose Concentration range	30- 50 mg/dL (1.7-2.8 mmol/L)	51-110 mg/dL (2.9-6.1 mmol/L)	111- 150 mg/dL (6.2-8.3 mmol/L)	151-250 mg/dL (8.4-13.9 mmol/L)	251-400 mg/dL (14.0 - 22.2 mmol/L)
Average	49.5 mg/dL (2.75 mmol/L)	103.4 mg/dL (5.74 mmol/L)	133.3 mg/dL (7.41 mmol/L)	196.1 mg/dL (10.89 mmol/L)	349.1 mg/dL (19.39 mmol/L)
Standard Deviation	1.6 mg/dL (0.09 mmol/L)	3.5 mg/dL (0.19 mmol/L)	5.0 mg/dL (0.28 mmol/L)	5.5 mg/dL (0.31 mmol/L)	13.4 mg/dL (0.74 mmol/L)
Coefficient of Variation	3.3%	3.3%	3.8 %	2.8 %	3.8 %

*The Dario® USB-C meter is equivalent to the Dario® LC and can be used with Android and iPhone SMDs.

Technical Specifications

Meter Specifications	
Result Range	20-600 mg/dL (1.1-33.3 mmol/L)
Calibration	Plasma-equivalent
Sample	Fresh capillary whole blood
Test Sample	0.3 microliter
Test Time	6 seconds
Assay Method	Glucose Oxidase Biosensor
Hematocrit Range	20% - 60%
Operating Temperature	50°F-95°F (10°C-35°C)
Operating Relative Humidity	15-85%
Altitude	Up to 10,000 feet (3,048 meters)
Memory	Unlimited results. Memory is limited to smart mobile device memory
Power Source	From the smart phone or mobile device
Meter Size	1.3 x 0.63 x 0.39 in (32.9 x 16.1 x 9.9 mm)
Weight	1.4 oz. (40 grams)

Meter Specifications	
Meter Lifetime	5 years
Protection degree from ingress of water or particulate matter	IP 21
Part number	1021-04 (Lightning) 1021-06 (USB-C)



Do not store near solvents and ionizing radiation sources.

Compliance with Electrical and Safety Standards

The Dario Blood Glucose Meter has been tested for:

- Electrostatic discharge as specified in IEC 61000-4-2.
- Immunity to radio frequency interference at the frequency range and test levels specified in IEC 61000-4-3.
- Electro-magnetic emission requirements as per EN 61326.
- EMC and is in compliance with IEC 60601-1-2.

1) Electromagnetic Emissions:

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The Dario System uses RF energy only for its internal function. Therefore, their RF emissions are very low and are not likely to cause any interference in nearby electronic equipment
RF emissions CISPR 11	Class B	The Dario System is suitable for use in all establishments,

		including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	NA	Not applicable (battery powered equipment)
Voltage fluctuations / flicker emissions IEC 61000-3-3	NA	

2) Electromagnetic Immunity:

Immunity test	IEC 60601 level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	8 kV contact 15 kV air	8 kV contact 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 5 %.
Electrical fast transient / burst, IEC 61000-4-4	2 kV for power supply lines 1 kV for SIP/SOP lines	Not applicable (battery powered equipment)	Mains power quality should be that of a typical commercial or hospital environment
Surge, IEC 61000-4-5	1 kV line to line	Not applicable	Mains quality of power should be that

Immunity test	IEC 60601 level	Compliance level	Electromagnetic environment - guidance
	2 kV line to earth	(battery powered equipment)	of a typical commercial or hospital environment.
Voltage dips and interruptions on power supply input lines IEC 61000-4-11	0 % UT for 0,5 cycle 0 % UT for 1 cycle 70 % UT for 25/30 cycles 0 % UT for 250/300 cycles	Not applicable (battery powered equipment)	Mains power quality should be that of a typical commercial or hospital environment. If the user of the equipment requires continued operation during power mains interruptions, it is recommended that the equipment be powered from an uninterruptible power supply or a battery.
Power frequency magnetic field, IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE: UT is the AC mains voltage prior to application of the test level.			

3) Electromagnetic Immunity for Home-health Environment

Immunity test	IEC 60601 level	Compliance level
IEC 61000-4-6 Conducted RF	3 Vrms 150 kHz to 80 MHz 6 Vrms in ISM bands (6.765 MHz to 6.795 MHz; 13.553 MHz to 13.567 MHz; 26.957 MHz to 27.283 MHz; and 40.66 MHz to 40.70 MHz) and amateur bands (1.8 MHz to 2.0 MHz, 3.5 MHz to 4.0 MHz, 5.3 MHz to 5.4 MHz, 7 MHz to 7.3 MHz, 10.1 MHz to 10.15 MHz, 14 MHz to 14.2 MHz, 18.07 MHz to 18.17 MHz, 21.0 MHz to 21.4 MHz, 24.89 MHz to 24.99 MHz, 28.0 MHz to 29.7 MHz and 50.0 MHz to 54.0 MHz)	Not applicable (battery powered equipment)

Immunity test	IEC 60601 level	Compliance level
IEC 61000-4-3 Radiated RF	10 V/m 80 MHz to 2.7 GHz	$[E] = 10 \text{ V/m}$
Proximity fields from RF wireless communications equipment	385 MHz	27 V/m
	450 MHz	28 V/m
	710 MHz	9 V/m
	745 MHz	
	780 MHz	
	810 MHz	28 V/m
	870 MHz	
	930 MHz	
	1720 MHz	28 V/m
	1845 MHz	
	1970 MHz	
	2450 MHz	28 V/m
	5240 MHz	9 V/m
	5500 MHz	
	5785 MHz	
IEC 61000-4-39 Immunity to magnetic fields in close proximity	8 A/m 30 kHz 65 A/m 134.2 kHz 7.5 A/m 13.56 MHz	8 A/m 30 kHz 65 A/m 134.2 kHz 7.5 A/m 13.56 MHz

Disposing of Used Products

The product may come into contact with blood during measurement. Used products (i.e., test strips and lancets) therefore carry a risk of infection.

Please dispose of used products according to the regulations applicable in your country. For information about correct disposal, please contact

your health care provider or your local council/authority.

The product falls outside the scope of the European Directive 2002/96/EC (Directive on waste electrical and electronic equipment (WEEE)).

Information Security and HIPAA

Dario smart mobile device App has been tested within the iOS and Android development environment and has been determined to be malware free. The Dario meter shipped by the manufacturer is malware free. Dario glucose meter device does not connect to the mobile device wirelessly. Users are required to unlock their smart mobile device prior accessing the Dario App to take a glucose measure test, as such they are assured of ongoing authorization access.

The Health Insurance Portability and Accountability Act of 1996 ("HIPAA") privacy rule provides guidance on security standards for the protection of EPHI (Electronic Protected Health Information), Labstyle Innovation Ltd. has performed a risk analysis and has implemented the standard and applicable security safeguards and is compliance with the HIPAA Security Rule and HITECH Act requirements for breach notification. Labstyle Innovation Ltd. ongoing compliance program includes policies and procedures for secure cloud operations.

Labstyle Innovation uses commercially reasonable standards of technology and operational security to protect users' glucose measurements data within the Dario App and cloud servers. Users data is stored and transmitted through the Dario App and/or web

portal to and from the remote server in encrypted form.

We are committed to protect the security, confidentiality and privacy of personal data we collect from our users. To learn more about how we comply with GDPR, HIPAA and other Information Security regulations, please read our Privacy Policy at <https://mydario.com/privacy-policy/>.

CHAPTER 9

WARRANTY, REPAIRS AND SUPPLIES

This chapter provides repair, warranty, service support, and ordering information for your Dario System.

Repair Policy

When under warranty, repair and service must be performed by LabStyle Innovation Ltd. When the LabStyle Innovation Ltd. warranty is not applicable, repairs may be performed by LabStyle Innovation Ltd. on a parts and labor basis.

After replacement, verify that the unit is fully functional by performing the normal setup procedure.



Do not remove the covers of the device components. Only perform maintenance procedures specifically described in this user guide.

Manufacturer's One Year Warranty

It is important to LabStyle Innovation Ltd. that you are completely satisfied with your Dario System. LabStyle Innovation Ltd. warrants that your new Dario will be free from defects in materials and workmanship for a period of one (1) year from the date of the original purchase. If during this time the Dario does not work properly because of a defect in materials or workmanship, LabStyle Innovation Ltd. agrees to replace or repair, free of charge, any and all parts proven to be defective and subject to warranty. This warranty is in lieu of all other warranties, expressed or implied, including any implied warranty of merchantability or fitness for

any purpose, other than stated herein. This warranty does not apply to the performance of a Dario System that has been damaged by accident or has been altered, misused, tampered with, or abused in any way. This warranty only applies to the original purchaser of the meter and/or its agents.

Supplies and Accessories

Dario System supplies and accessories are available from LabStyle Innovation Ltd. or from customer support.

Item	Catalog Number
System	1138-02 – Dario Welcome Kit (Lightning) 1138-05 – Dario LC Welcome Kit 1152-02 – Dario Welcome Kit (USB-C) 1152-05 – Dario USB-C Welcome Kit 1147-02 –Dario Starter Kit (Lightning) 1147-05 –Dario LC Starter Kit 1200-05 – Dario USB-C Starter Kit 1264-05 - Dario USB-C Welcome Kit (for iPhone and Android) 1265-05 - Dario USB-C Starter Kit (for iPhone and Android)
Test Strip Cartridge	1105-01 / 1120-05 – Dario Test Strip (1 x 25)
	1024-01 / 1024-05 – Dario Test Strip (2 x 25)
	1074-01 / 1074-05 – Dario Test Strip (4 x 25)
Lancet	1018-01 / 1121-05 – Dario Lancet (1 x 100)
Control Solution	1054-01 / 1054-05 – Dario Glucose Control Solution (2x2.5 ml)
Disposable Covers (available for the US market only)	1128-05 – Dario Disposable Covers (1 x 100)

Item	Catalog Number
Software	1022-01 – Smart mobile device application iOS 1022-02 - Smart mobile device application Android

Contact Customer Support

You may contact customer support with any questions or issues about Dario and its accessories using one of the following methods:

- By phone:
 - USA: toll-free Monday to Friday, 9:00AM – 9:00PM EST at 1-800-895-5921 or www.mydario.com
 - Canada: toll-free at 1-800-461-0991 or www.mydario.ca
- Through email by sending a message to support@mydario.com.

Please contact your health care provider for questions outside of the customer support hours.